

**Read and follow all instructions on box or packed with products before using.**

## **How it works**

The Aroma Season heating eye mask provides deep penetrating heat therapy for dry eye syndrome, MGD (meibomian gland dysfunction) and blepharitis, allows the glands to function properly to give immediate dry eye relief.

H -- turn off automatically after 20 mins at High temperature mode

M-- turn off automatically after 20 mins at Middle temperature mode

L -- turn off automatically after 60 mins at Low temperature mode

After using it, your eyes will feel refreshed and rejuvenated.

## **How to use**

1. Insert the DC plug into the porter on eye mask
2. Connect the USB plug on any 5V/1-2A Power Supply
3. Spray a little warm water on the eyes side of the mask to avoid dry
4. Use the controller to select a temperature mode

## **Cleaning**

- ◆ Surface clean only with a damp cloth.
- ◆ Do not use soap or detergent as this can contaminate the mask.
- ◆ Allow the mask to dry completely before heating again.

## **Warning**

- ◆ Do not use while sleeping
- ◆ This eye mask is not allow to use on an invalid, sleeping or unconscious person, or a person with poor blood circulation.
- ◆ Never pull this eye mask by the supply cord. Burns will result from improper use