Heated Eye Mask

Owner's Manual

Thank you for your purchase of this product. This manual describes how to use your Heated Eye Mask. Be sure you have read and understood its contents before using it.

Specifications

Input:	AC 100-240V 0.4A 50/60Hz	Output:	3-5V/Max 1A
Material:	100% cotton	Dimension:	8.5*4.0 inches
Temperature Mode:	High/Medium/Low		

It is recommended that you use your mask for 15-20 minutes each time, which is an optimal heating time for eye treatment.

How to use

1 Plug the DC Connector on Controller into the DC Port on Eye Mask.

2 Plug the USB Connector on controller into a 5V 1-2A USB outlet.

3 Press the Power Button to turn it on and select a temperature mode (High/Medium/Low) or turn off the heating. Without any operation, it will stop heating automatically for safe case.

High ----- stop heating in 20mins
Medium ----- stop heating in 40mins
Low ----- stop heating in 40mins

4 If your eyes feel dry when you use it, spray 2-3 drops of warm water on the eyes side of the eye mask for moist heating.

Note: There is no battery inside the eye mask. Keep it power on when using it.

Wash Care

Wash the eye mask by hand and do not twist it. Let the eye mask air dry completely before you use it next time.

Important Safety Warnings

- Do not use on high setting while sleeping.
- This eye mask is not to use on an invalid, sleeping or unconscious person, or a person with poor blood circulation.
- Never pull this eye mask by the supply cord. Do not fold while heating. To avoid overheat, do not put it under the pillow and comforter. Burns will result from improper use.

Warm Tips

How warm it feels depends on the environment temperature. The environment temperature difference could be more than 50°F between winter and summer. It's suggested to use high mode on cold days and low mode on hot days. Stop heating when you feel too hot. Be care of moderate temperature burn.

What's a moderate temperature burn?

Even at moderately low temperature (104-140°F), when in contact the same part of the skin for long periods of time, moderate temperature burns may occur, which result in red spots or blisters. If there are signs of moderate temperature burns, see a doctor immediately.

This product is not intended for diagnosing or curing any diseases, and is not a replacement for consulting. If symptoms do not improve, please consult a health care professional.

AROMA SEASON INC
Email: info@aromaseasoninc.com
Address: 1150 S MILLIKEN AVE. #1061
ONTARIO CA 91761

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